

# RESCUE SHEETS

## Instruction Manual



**Important notice:**  
STUDY THIS MANUAL BEFORE USE!

# General information

## Two types of rescue sheets



Rescue Sheet - Compact  
Comes with a yellow strap and a karabiner



Rescue Sheet - With PVC handles  
Comes in a carrying bag

## Application

The Rescue Sheet is designed to make it easier to effectively transfer patients with minimal effort. With 12 secure carrying handles, multiple rescue responders can effortlessly balance and carry a patient from the scene quickly. The patient's weight is evenly distributed across the canvas sling allowing for minimal disruption even over rough terrain.

## Safety

The Rescue Sheet may be operated only after consulting the instruction manual and by professional qualified personnel.

Since defects cannot be excluded due to wear, the Rescue Sheet is recommended to examine for visible damage before and after each use. Tetcon Global BV and its distributor is not responsible for damage caused by improper handling by unauthorized locations.

## Maintenance

The Rescue Sheet can be wiped off with a cleaning detergent. You may also wash it in the machine at max. 60°Celsius. Do not bleach. Do not use washing powder with a fabric softener. Please only dry on a low temperature. Do not iron.

## User responsibility

The responsibility for the use of the Rescue Sheets lies with the user.

## Technical specifications

Size Rescue Sheet: 200×78 cm / 81" – 31" (complies with DIN EN 1865-1)

Size feet section: 40×78 cm / 15" – 31"

Weight Rescue Sheet: 2kg (complies with DIN EN 1865-1)

Material top side: PU coating


Material bottom side: 100% polyester

Weight capacity: 500 kg / 1100 lbs (complies with DIN EN 1865-1)

# Layout



- 1. Handle (12x)
- 2. Flap for patient's feet
- 3. Sheet for carrying the victim
- 4. Head section

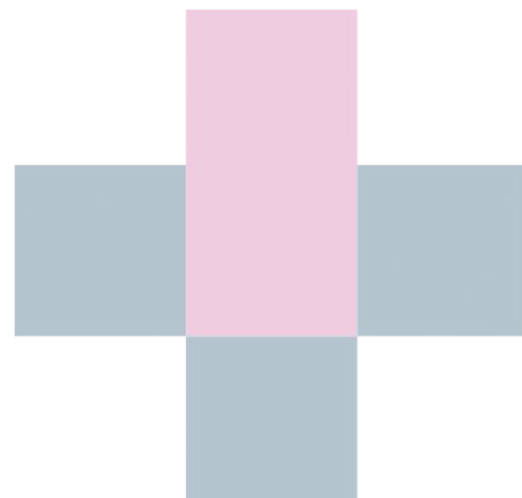



# Guideline for using the Rescue Sheet



Position the victim in the middle of the Rescue Sheet.

If necessary, you may use the logroll-technique to place the Rescue Sheet underneath the victim.



When you operate the Rescue Sheet by yourself, you may slide the victim. Use the handles for extra grip.

Make sure that the arms stay on the canvas.



When you operate the Rescue Sheet with two people, one person should stand at the feet end and one person should stand at the head end.

Make sure to keep your back in a straight position to avoid injuries. Go through your knees before lifting.



In an ideal situation, operate the Rescue Sheet with 3 or 4 users.

Use the handles on the side of the Rescue Sheet to lift up the victim.

