



Your Retmex Care Guide

LEAVE NO ONE BEHIND

THE RETMEX COLLECTION

Nomex® Model

93% Nomex® | 5% Kevlar® | 2% P-140

Fire Retardant Webbing

Storage system: Strap + Carabiners

Comfort Model

100% Polyester + PU

Fire Retardant Webbing

incl. PVC on Handles


Storage System: Portable Bag

Compact Model

100% Polyester + PU

Fire Retardant Webbing

Storage System: Strap + Carabiner

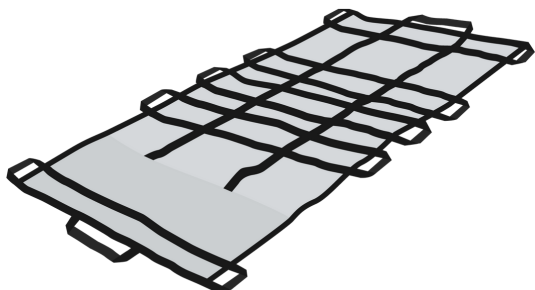

STORAGE & CARE

Storage

Once the rescue sheet is properly dry, you can stow it away in its bag or fold it away. Store your Retmex in a dry place.

After use

Clean your Retmex with soap and wipe off the canvas or machine wash using the instructions per model below.

Always check for damages before storing your Retmex. Damages to your Retmex can influence the safe use of the rescue sheets.

MACHINE WASH YOUR RETMEX

Nomex® Model

Wash at 60°C

Tumble dry - low heat

Iron at low temperature

Do not bleach

Comfort Model

Wash at 60°C

Tumble dry - low heat

Iron at low temperature

(Do not iron PVC handle)

Do not bleach

Compact Model

Wash at 60°C

Tumble dry - low heat

Iron at low temperature

Do not bleach





**Thank you for being
part of our journey.**