

Learn the safe use of

'*The Toughest Evacuation Mat*'



Proprietary notice

The information disclosed in this manual is the property of Tetcon Global BV, Eindhoven, The Netherlands. Tetcon Global BV reserves all patent rights, proprietary design rights, manufacturing rights, reproduction use rights and sales use rights thereto, and to any article disclosed therein except to the extent those rights are expressly granted to others or where not applicable to vendor proprietary parts.

Company information

Tetcon Global BV
Ruysdaelbaan 3G
5642 JJ Eindhoven
The Netherlands
www.s-capeplus.com
www.tetcon-ge.com

User information

S-CAPEPLUS Premium comes with a comprehensive 'User Manual', for the correct usage protocol. Tetcon Global BV cannot be held responsible for equipment that has not been maintained in accordance with its guidelines, specification or for the incorrect use of equipment under working conditions.



Incredibly powerful. Designed with the world's strongest fibre, Dyneema®.*
Leave no one behind during an emergency evacuation.



The strength of Dyneema® is insane. It's 15 times stronger than steel.
By adding this powerful component to this premium evacuation mattress, it can be used for more
purposes and different types of terrain.

What will you learn?

In this eBook, you will learn the basics techniques that are required to operate the Evacuation Mattress S-CAPEPLUS Premium.

1. The safe usage

In this eBook, each user step is carefully described to ensure the safe usage.

2. Teach your colleagues

By closely observing the steps in this eBook, you will be able to teach your colleagues the safe usage.

3. Avoiding mistakes

It is important to avoid user mistakes. Therefore, it is recommended to reread each step before taking action.

The Evacuation School

To ensure the safe use of the entire S-CAPE range of evacuation mats, sheets, mattresses and sledges, we decided to start an Evacuation School.

This is the Only Digital Evacuation School. It is completely free. It allows you to practice where and whenever you want.

Choose the device you want to train with. See how easy it is, then try it yourself.



[To The Evacuation School](#)

Let's get started.

Learn the safe use of
S-CAPEPLUS Premium
Evacuation Mattress
in 7 easy steps.

Scroll down





STEP 1

Remove the S-CAPEPLUS Premium from the carrying bag, the Wall Cover or the storage cabinet.



[Video 'How to fixate the yellow strap'](#)

STEP 2

Position the patient in the middle of the mattress. Make sure the positioning belt is placed right around the person's torso by pulling the belt through the ring on the right hand side. Fix this in place with the hook and loop fasteners. Position the arms along the body and above the belt.



STEP 3 (OPTIONAL)

Adjust the feet board to the person's length, so this makes it ideal for people with a shorter stature. The feet board provides extra stability to the person on the evacuation mattress. You can attach the Feet Board to the clips at different heights. The Feet Board can be purchased additionally with your S-CAPEPLUS Premium





[Video 'How to use the body flaps'](#)


STEP 4

Now firmly pull the long flaps towards each other. Connect and stick the right flap to the left strap with the hook and loop fasteners. Close this up as tightly as possible. Now close up the flap at the foot end. Again, do this as tightly as possible.



STEP 5

All elements of the head blocks can easily be adjusted with the hook and loop fasteners. Position two elements of the head blocks next to the head. Place them above each other. Do this on both sides. Head and chin straps hold the head securely in place.

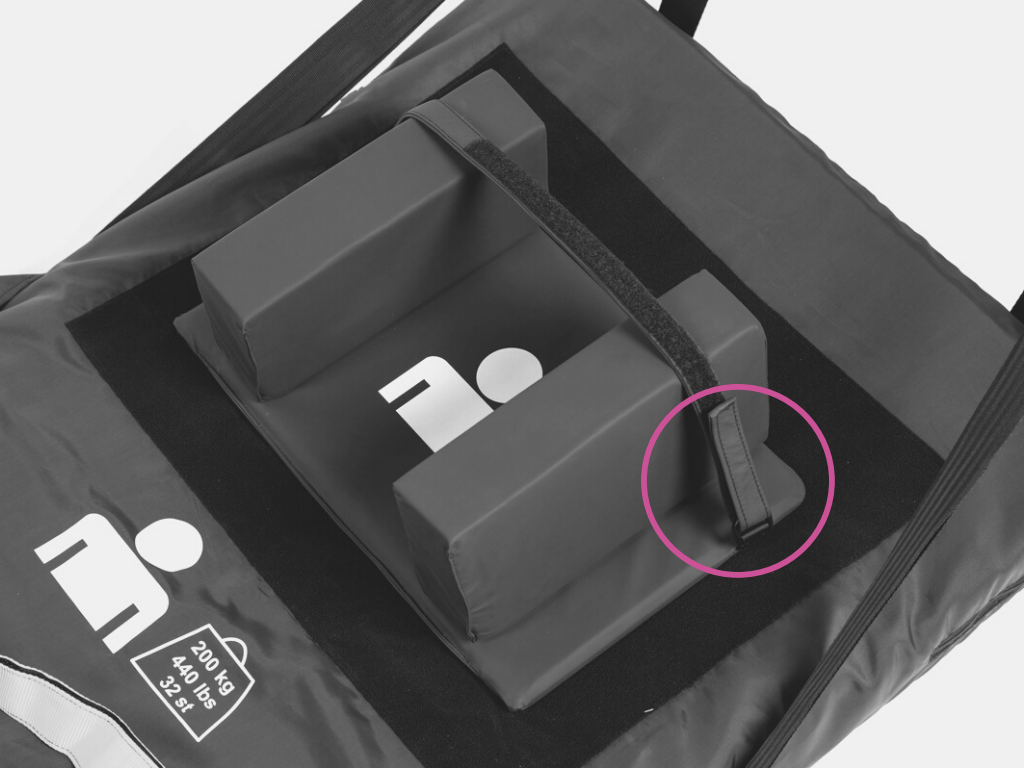

STEP 5 (OPTIONAL)



All elements of the head blocks can easily be adjusted with the hook and loop fasteners. Position two elements of the head blocks next to the head. Place them above each other. Do this on both sides. Head and chin straps hold the head securely in place.



[Video 'How to make short turns with Evacuation mat'](#)

STEP 6

Take a firm hold of the loop at the foot end, allowing you to evacuate the patient in a dragging motion. A second rescuer can use the loop at the head end to navigate more easily around bends.



STEP 7

If you find yourself on a higher floor, you will need to use the stairs to make your way downstairs, as you won't be able to use any possible lift present during a fire.

This will not present any problems, providing you take command of the stairs. Make sure you have enlisted the help of a second rescuer to hold the loop at the head end. This person can be responsible for any necessary braking when making your way down the stairs.

Position yourself a few treads lower and pull the patient toward you. Carefully lower the mattress once the patient has crossed the stairwell with his/her buttocks. Make sure you keep a firm hold of the railing at all times and always maintain contact with the S-CAPEPLUS.

Don't miss out any treads when going down the stairs and gradually do so step by step.

The fast and safe way to evacuate immobile people. Can be easily dragged by just one rescuer.

Safety information

These instructions contain information concerning the safe use of the S-CAPEPLUS. To ensure that the S-CAPEPLUS is used correctly and safely, this information must be read prior to placement and usage of the S-CAPEPLUS.

- Always observe the instructions given in the instruction manual when using the S-CAPEPLUS.
- Only use the S-CAPEPLUS in emergencies, such as a fire, power cut, gas leak terrorist attack, natural disaster, extreme smoke or any other life-threatening situation or emergency.
- Product training with the S-CAPEPLUS must always be given by a certified emergency response team member.
- The use of alcohol and/or prescription and over-the-counter medicine may have a negative effect on the skill of the rescuer who pulls the S-CAPEPLUS.
- Do not remove any parts of the S-CAPEPLUS.
- Make sure the safety information and instruction manual are kept close to where the S-CAPEPLUS has been installed.
- This S-CAPEPLUS is not intended for recreational use.
- Do not use flammable vapours, materials or liquids, such as petrol, alcohol, etc., close to the S-CAPEPLUS.
- Not observing the instructions given in the instruction manual provided with the S-CAPEPLUS may result in serious physical injury or even death.
- sharp edges or protrusions may damage the S-CAPEPLUS and, therefore, reduce the effectiveness of the product.
- Practice using the S-CAPEPLUS at least once a year in order to guarantee its correct and safe use in event of an emergency.
- Contact the various rescue services (fire brigade, ambulance service, etc.) to inform them of the purchase and presence of the S-CAPEPLUS.
- The S-CAPEPLUS is not suitable to be pulled over a naked flame.
- The S-CAPEPLUS must be carefully checked for damage after it has been used. If damaged is observed, the S-CAPEPLUS must not be reused. Furthermore, the S-CAPEPLUS must be folded up and stored away each time after use.
- The S-CAPEPLUS has a 1-year warranty for material and manufacture faults, which is valid from the date of purchase. The manufacturer may ask you for proof of the purchase date. Therefore, it is recommended to keep the invoice in a safe place. Faults or problems with the use of the S-CAPEPLUS as a result of an accident, misuse, incorrect use, alterations, vandalism, incorrect placement, incorrect maintenance or administration, as well as overdue normal and routine maintenance, are not covered by the warranty.
- The manufacturer is not liable for (consequential) damage as a result of the use of the S-CAPEPLUS.

Never overestimate yourself! Evacuation via a staircase can be very dangerous! You will have to support a heavy weight and you must be certain that you are capable and strong enough to keep this weight under control all the way down the staircase. If in doubt as to whether you are able to evacuate somebody down a staircase yourself, you must always ask a second care worker to help you or leave the task to someone else. Your own safety is of the utmost importance! Regularly practicing using the S-CAPEPLUS during evacuation exercises and always observe the instructions given below.

Only use in case of emergency situations. The manufacturer and supplier cannot be held liable for any improper use!

TUTORIAL & EBOOK COMPLETE

