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Produttore:

Baby-Mover B.V.
Poppenbouwing 1
4191 NZ Geldermalsen
Paesi Bassi

Instructions for the use of the NEWBORN BABY MOVER® from Baby-Mover B.V.

IMPORTANT! READ CAREFULLY AND KEEP FOR FUTURE REFERENCE.

WARNING :

- Your balance may be adversely affect by your movement and that of the child(ren)
- Take care when bending or leaning forward or sideways;
- This carrier is not suitable for use during sporting activities.

Use only for its intended purpose:

The NEWBORN BABY MOVER® is a professional evacuation apron intended for the emergency removal of babies to a place of safety. In case of fire or other emergencies, up to 3 babies with a weight up to a maximum of 5kg (11 lb) per baby can be evacuated from buildings or other structures. The NEWBORN BABY MOVER® is neither intended nor suitable for any other purpose than that described here.

Instructions for use:

A. Preparation :

- Remove the NEWBORN BABY MOVER® from its packaging and hang it up at a known place that is quickly and easily accessible in case of emergency
- Protect the NEWBORN BABY MOVER® against moisture, heat, strong sunlight and ensure that the pockets remain empty. This will prolong the life of the product.
- Keep the NEWBORN BABY MOVER® out of reach of children and animals
- Baby-Mover B.V. accepts no liability for damage caused by improper use of the NEWBORN BABY MOVER® or use of the NEWBORN BABY MOVER® for any purpose other than that for which it is intended.
- As the NEWBORN BABY MOVER® may be used monthly at evacuation drills, some signs of wear and tear may appear after a few years. Therefore, check regularly that the NEWBORN BABY MOVER® has no defects or excessive wear.
- Report any defect without delay to Baby-Mover B.V. and never try to repair it yourself!
- If possible, practice the scenario described in section B at least once a month

B. If evacuation becomes necessary:

- The rescuer first places her/his head and arms through the holes in the NEWBORN BABY MOVER®, with the pockets to the front. Place the babies in the carrying pockets as detailed below. Try to evenly distribute the weight over the three pockets.
- Place the baby horizontally with the head on the side of the black mesh in the carrying pocket. Turn the face of the baby to the open woven mesh from the carrying pocket of the NEWBORN BABY MOVER®.
- Close the carrying pocket with the clip fastener. Adjust this clip fastener looser or tighter.
- The NEWBORN BABY MOVER® is made from fire resistant material. Therefore flammable material should as little as possible be allowed either on the NEWBORN BABY MOVER® or in the carrying pockets.
- Pay full attention to the children whilst they are being carried.
- Do not use (hot) drinks or food while carrying children.

Washing instructions:

Guarantee:

Baby-Mover B.V. guarantees the proper function of the NEWBORN BABY MOVER® and the suitability of the materials used for a period of five years from the date of sale. The NEWBORN BABY MOVER® is manufactured with great care. Both the materials and the construction have been extensively tested for their quality and suitability. The warranty is valid only if the instructions for use shown above are observed. In case of defects Baby-Mover B.V. has the right to take the decision whether to repair the NEWBORN BABY-MOVER®, replace it, or to repay the purchase price. Complaints regarding any deficiencies must be reported in writing by post to Baby-Mover B.V. within 5 working days after discovery, the affected NEWBORN BABY MOVER® must accompany the letter.

The warranty will be forfeited in cases of improper or inappropriate usage. The warranty is also forfeited if modifications and repairs to the NEWBORN BABY MOVER® are carried out by persons other than staff of Baby-Mover B.V.

Liability

Baby-Mover B.V. shall not be liable for damage resulting from improper use of the NEWBORN BABY MOVER® or use of the NEWBORN BABY MOVER® for any purpose for which it is not intended.

Baby-Mover B.V. shall not be liable for visible defects, which have not been reported to the company in due time.

Manufacturer:

Baby-Mover B.V.
Poppenbouwing 1
4191 NZ Geldermalsen
The Netherlands